# ICE-VIBE

CIRCULATION THERAPY

## USER INSTRUCTIONS & SAFETY GUIDE

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***Each Ice-Vibe pack or knee pack contains:***

- ✓ 2 boots or wraps
- ✓ 2 cold packs
- ✓ 2 integrated vibrating panels
- ✓ 1 battery charger and 4 universal adapters
- ✓ Spare parts sold separately

***Each Hock wrap pack contains:***

- ✓ 2 wraps
- ✓ 4 cold packs
- ✓ 2 integrated vibrating panels
- ✓ 1 battery charger and 4 universal adapters
- ✓ Spare parts sold separately

ICE-VIBE bag

2 vibrating panels with integrated battery controller

2 cold packs

1 battery charger and 4 adapters for universal use

Ice-Vibe Boots

Ice-Vibe Hock Wrap

Ice-Vibe Knee Wrap

Ice-Vibe Xfull Boot

## Cold Packs

## Hock Ice Packs

## Knee Ice Packs

## Cool Bag

## Integrated Vibration Panels & Battery

## Cable & Transformer

## *How do they work*

Ice-Vibe boots are rechargeable vibrating boots that boost circulation in horse's legs. They combine the effects of cooling and massage therapy to effectively treat strains, ligament and tendon damage. The boots can be used with and without cold packs to reduce swelling and stimulate blood flow. Cooling slows down blood flow to prevent swelling, while the massage effect encourages lymph fluid to move, draining inflammation and stimulating repair to damaged tissue. Using the boots before exercise can help reduce the risk of injury as increasing blood flow to the tendons and ligaments makes them more elastic. Everyday wear and tear on joints can be managed with daily use of the boots, and rehabilitation through injury can be achieved more quickly.

## *Setting up your boots*

Insert vibrating panels into pockets flat side to the horse's leg. Use one panel in each boot. The battery/control unit is marked with an on/off switch and is integrated into the panel. **Place cold packs in fridge/freezer before using.**

Front of Ice-Vibe Boot

Side of Ice-Vibe Boot

ABOUT YOUR ICE-VIBE BOOTS

ICE-VIBE  
CIRCULATION THERAPY

## *Using your Ice-Vibe boot*

Place the cold packs on your horse's legs. Secure them with the velcro strip. Place the flared part of the cold pack to the front of the fetlock. The packs are clearly marked left and right. Place the boot over the top of the cold pack and secure with velcro straps. Switch the boot on by **holding down the on/off button for 3 seconds at the back of the boot where clearly marked on/off.**

Your boot has three settings. To change the setting press the on/off button once. Your boot will switch off after ten minutes on setting one and twenty minutes setting two and three. If you need to switch off your boot manually hold the on/off button down for 3 seconds.

## *Using your boots with cold packs*

Your cold packs can be cooled in a fridge or freezer for a minimum of 2 hours before use. Fridge may be adequate for day to day use. Always use your cold packs if swelling is present.

## *Using your boots with warm packs*

Horses suffering with joint stiffness may benefit from warm massage. You can heat the packs by soaking them in warm water. Do not microwave them. Do not soak in boiling water as you may burn your horse.

Always start your horse on level one, once a day for two to three days to allow him to adjust to the boots.

**Setting two is your main setting before and after exercise.**

## *How to charge the battery*

- 1 The battery is integrated into the top of the panel. It has been partially charged, but you may need to recharge it before you can switch on your device for the first time.
- 2 Connect the charger to a wall outlet.
- 3 Connect the charger to the battery. The battery is integrated into the panel; the connection is at the bottom of the panel.
- 4 Battery will fully charge in two hours with one battery or in four hours for two batteries simultaneously charging.
- 5 Disconnect the charger from the wall outlet when the charger is not in use. Your device can be charged from a usb port e.G. A laptop, when a wall outlet is not available. Should you require a second usb charger the Ice-Vibe battery requires a usb mini b size cable.
- 6 If you're going to store your boots for more than a month, store with battery fully charged. If stored and not charged the lifespan of your battery will be reduced.

**\* If your battery is not integrated, remove battery, connect charger and charge both batteries for 4 hours.**

## *Your boots should last for two hours once charged*

Always start your horse on the first level that switches off after ten minutes for the first three days until his legs get used to the Ice-Vibe boots.

### *What settings to use*

#### **Setting 1**

Always start on setting one for the first few days to allow your horse's legs to adjust to the wrap and to make sure the leg does not swell from use

Use for first few days of rehabilitation

#### **Setting 2**

Use to prepare for exercise without the cold packs

Use after exercise with cold packs

Use for rehabilitation

#### **Setting 3**

This is the strongest setting

Should only be used with cold packs

Should be used on larger boned horses or horses with old excess scar tissue

## PLEASE NOTE:

- When swelling and bruising is present always use with cold packs
- If your horse's legs swell after use s/he may have a fracture or an infection
- If you are unsure please consult your veterinarian

Always start your horse on setting one for the first three days to allow your horses legs to adjust to the boot and to make sure the leg does not swell from use. If your horse's legs swell after use your horse may have a fracture or infection.

The ice-vibe boots are designed to assist you to prepare your horses legs before exercise and to minimise soreness and bruising after exercise they are not a medical device.

## *How do they work*

Ice-Vibe hock and knee wraps help increase circulation to your horse's hocks and knees. However, they cannot be used on infection as the massage effect may spread the infection. Do not use on fractures in the first four weeks as it may irritate and cause some swelling. Ice-vibe wraps are designed to help you prepare your horse's legs before exercise and reduce inflammation, discomfort and bruising after.

**Remember: this is not a medical device.**

## *Setting up your hock / knee wraps*

- 1 Insert vibrating panels into pockets with the flat side to the horse's leg and the battery end of the panel going into the wrap first. Use one panel in each wrap.
- 2 The battery/control unit is marked with an on/off switch and is integrated into the panel. When inserted into wrap you should be able to see on/off button on outside of wrap.
- 3 The hock wraps give you the option of having the panel on the inside or the outside of the hock.
- 4 The panel for the knee wrap goes into the middle of the wrap.
- 5 Place cold packs in fridge/freezer for a minimum of two hours before using.


## *Turning on your Ice-Vibe wraps*

- 1 Switch the panel on by holding down the on/off button at the bottom of the wrap for 3 seconds.
- 2 Your wrap has three settings. It switches on at setting one. To change the setting press the on/off button once.
- 3 Your wrap will switch off after ten minutes on setting one, press the button once more for setting two, this lasts twenty minutes.
- 4 Press once more for setting three. This also lasts 20 mins.
- 5 If you need to switch off your wrap manually hold the on/off button down for 3 seconds.


Always start your horse on level one, once a day for a couple of days to allow him to adjust to the wraps.

**Setting two is the best setting for before and after exercise.**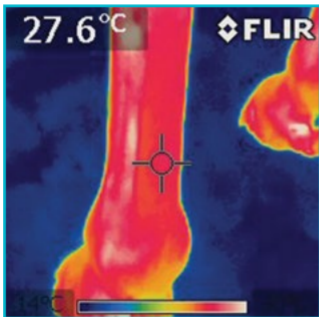

BEFORE THERAPY


Resting normal leg  
before therapy.

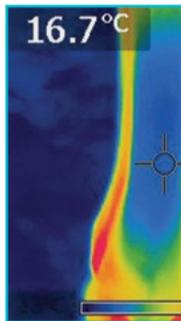AFTER USING ICE  
& VIBRATION THERAPY

Tendon after using ice  
& vibration with Ice-Vibe  
significantly cooler.


ICE PACK  
APPLY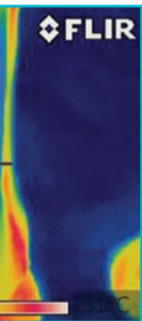

Ice pack and  
Ice boot. Leg is

AFTER THERAPY USING  
ONLY VIBRATION

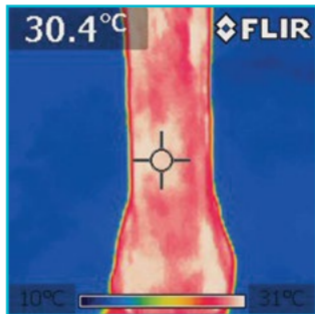PREVENTS INJURIES:  
Leg after using vibration panels only with  
Ice-Vibe boot. Leg warmed up and ready for  
exercise.

ICE-VIBE  
CIRCULATION THERAPY

THERMAL PHOTOGRAPHS

## *Fitting your Ice-Vibe knee wrap*

- 1 Place the wrap around your horse's knee, with the massage panel sitting at the front middle of the knee and the back of the knee coming out through the gap at the back of the boot.
- 2 Secure the top velcro strap above your horse's knee first.
- 3 Fasten the outer elasticised strap firmly over the top of the first strap. (see picture).
- 4 Ensure back of the knee is exposed (see picture).
- 5 Secure the bottom strap below the knee loosely.
- 6 When treatment is finished remove the wrap.

## *Fitting your Ice-Vibe knee wrap with cold pack*

- 1 To attach the cold pack to the inside of the knee wrap place it inside the front of the wrap, with the velcro piece facing the outside of the wrap.
- 2 Then fold the velcro down over the top to secure it.

As the cold pack is applied directly onto the knee we advise leaving it out of the freezer for 5 minutes prior to use. Your cold packs can be cooled in a fridge or freezer for a minimum of 2 hours before use. Always use your cold packs if swelling is present.

Insert vibrating panel into pocket

Making sure recharge point fits to bottom of pocket enlign with ON/OFF switch on opposite side

Insert cold pack to inside of wrap

Attach velcro to outside of wrap

### *Fitting your Ice-Vibe Hock wrap (your wraps are clearly marked left and right)*

- 1 Place the wrap around your horse's hock with the point of the hock sitting through the hole in the wrap.
- 2 Secure the top Velcro strap above your horse's hock.
- 3 Then secure the second strap firmly.
- 4 Then secure the third strap firmly.
- 5 Secure the bottom strap below the hock firmly.
- 6 When treatment is finished remove the wrap.

### *Fitting your Ice-Vibe Hock wrap with cold packs*

The hock wrap comes with two cold packs that are the same size. The packs fit into the two pockets that sit either side of the hock. Cold packs can be cooled in a fridge or freezer for a minimum of 2 hours before use. Always use your cold packs if swelling is present.

### *Using your Ice-Vibe wraps with warm packs*

Horses suffering with joint stiffness may benefit from warm massage. You can heat the packs by soaking them in warm water. Do not microwave them. Do not soak in boiling water as you may burn your horse.

Insert vibrating panel in pocket behind cold pack

Insert cold packs either side of Hock Wrap

When fitting wrap, close straps numerical order as shown

When fitting wrap, close straps numerical order as shown

When fitting wrap, close straps numerical order as shown

Each pair has a left and right cold pack

Flared end to cover the Fetlock Joint

## *User guide safety: Ice-Vibe circulation wraps*

- Only qualified personnel may repair this product.
- Keep your device dry it is not water resistant.
- Do not store the electronic components in extreme hot or cold temperatures.
- Do not drop, knock or shake device.
- Do not allow horses to chew or stand on device as it contains delicate circuit boards.
- Use battery and charger for their intended purposes only. Improper use, or use of unapproved batteries or chargers may present a risk of explosion or other hazard and will invalidate warranty.
- Never use a damaged battery or charger.
- Do not place cold packs in a microwave.
- Do not heat cold packs.

All electrical and electronic products and batteries must be taken to separate collection at the end of their working life. This requirement applies in the European Union. Do not dispose of these products as unsorted waste.

## *User guide safety: Ice-Vibe circulation wraps*

The warranty period begins from the time of original purchase by the first end user. The electronics have a 6 month warranty or 1000hrs. The limited warranty period will not be extended or renewed due to subsequent resale. This limited warranty does not cover defects caused by rough handling or chewing from the horse, defects caused by sharp items or damage caused by misuse of the product. This limited warranty does not cover defects or damaged cause by the fact that the product was connected or used with any product not supplied by Horseware®. This limited warranty is not enforceable if the product has been exposed to dampness, extreme thermal or environmental conditions, or to spillage of food or liquid or chemical products. This limited warranty is not enforceable if the product has been modified or repaired by an unauthorized person. This limited warranty is not enforceable if the battery has been opened, broken or shows evidence of tampering.

To view our other ICE-VIBE products or to make an enquiry contact us via our website **[www.ice-vibe.com](http://www.ice-vibe.com)** or email us at **[ice-vibe@horseware.com](mailto:ice-vibe@horseware.com)**

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